MAC™ 2000

PDF Reports

Software Version 1.1
2053535-072 Revision A
15 May 2015

This document describes how to configure the MAC 2000 ECG Analysis System to generate PDF reports in a network environment and provides the following information:

- Required Setup
- Required Information
- Instructions for configuring PDF Reports

Required Setup

Before beginning the configuration, verify the following setup has been completed:

- The **PDFC** option has been enabled on the MAC 2000 (**System Configuration > Options Setup**).
- For wired connections: the **LANC** networking option has been enabled on the MAC 2000 (**System Configuration > Options Setup**) and a network cable has been connected to the device.
- For wireless connections: the **LANC** and **WIFC** networking options have been enabled on the MAC 2000 (**System Configuration > Options Setup**) and a wireless dongle has been connected to the device.
- A shared directory has been set up on the customer's network.
- The customer's DNS server has been configured to resolve share names and, if the network spans multiple subnets, WINS has been enabled. (The MAC 2000 cannot be configured to use the IP address of the server where the shared folder resides.)

For more information about setting up your system, refer to the *MAC™ Resting ECG Systems LAN Option to CardioSoft/CS Connectivity Server Installation and Troubleshooting Guide* (2040396–104).

Required Information

Before beginning the configuration, obtain the following information from the customer:

- **Share Name**
This includes the path for the shared directory.
- **User Name**
This must be a user account with permissions to write files to the shared directory.
- **Password**
This must be the logon password of the selected user account.
- **Domain Name**
This is required only if the user account resides on a domain.
- **IP Addressing**
You need to know whether the customer uses automatic IP addressing via DHCP or static IP addressing. If using static IP addressing, you need to obtain the IP, Netmask, Gateway, DNS, and WINS addresses of the MAC 2000 device.

Configuring the PDF Reports

After verifying the required setup has been completed and obtaining the necessary configuration information, you are ready to begin configuring the system to generate PDF reports in a network environment.

The configuration process consists of the following tasks:

1. [“Configuring the Shared Directory”](#)
2. [“Configuring the Network”](#)
This will be one of the following procedures, depending on network type:
 - [“Configuring a Wired Ethernet Network”](#)
 - [“Configuring a Wireless Network”](#)
3. [“Configuring the PDF Report Name”](#)
4. [“Configuring Automatic Export to Shared Folder”](#)

The remainder of this document describes each task in detail.

Configuring the Shared Directory

In the MAC 2000 application, select **Main Menu > System Configuration > Communication Setup**. When the **Communication Setup** window opens, scroll to the **Shared Directory Settings** section.

Configure the shared directory settings as follows:

Enable this checkbox to activate **Shared Folder** functionality.

Enter the path for the shared folder. Use only the **Computer Name** and **Shared Directory Name**, in this format:
\\ServerName\ShareName
Maximum: 256 characters.
Important
Do NOT enter the computer IP address. The IP address does not work in this field; only the computer name does.

Enter the **Username** and **Password** of an account with permissions to the shared folder.
Re-enter **Password** in the **Confirm** field.
Username allows up to 30 characters.

If the user account is on a domain network, enter the user's **domain** name.
If the user account is not on a domain network, leave this field blank.

When you are done configuring the shared directory, proceed to “[Configuring the Network](#)” on page 4.

Configuring the Network

After configuring the shared directory, you can begin to configure the network. The procedure you use depends on your network type.

- If the **LANC** option was enabled, you will be using a wired Ethernet network. Proceed to “[Configuring a Wired Ethernet Network](#)” on page 4.
- If the **WIFC** option was enabled, you will be using a wireless network. Proceed to “[Configuring a Wireless Network](#)” on page 6.

Configuring a Wired Ethernet Network

On the **Communication Setup** window, page down until the **Wired (LAN) Settings** page is displayed. The settings vary depending on whether your network uses DHCP or static IP addresses.

If your network uses DHCP, configure the **Network LAN Settings** as follows:

The screenshot shows the 'Wired LAN Settings' window. At the top, there are two text boxes: 'Cardiograph Device Name' with the value 'GE_SDE0E210017NA' and 'Serial/IP Redirector Listen Port' with the value '3039'. Below these are two checked checkboxes: 'Obtain an IP address automatically (DHCP)' and 'Obtain DNS server address automatically (DHCP)'. Under the first checkbox, the following values are listed: IP Address: 192.168.0.114, Netmask: 255.255.255.0, and Gateway: 192.168.0.1. Under the second checkbox, the following values are listed: Preferred DNS Server: 192.168.0.1, Alternate DNS Server: 0.0.0.0, Preferred WINS Server: 0.0.0.0, and Alternate WINS Server: 0.0.0.0. The window has 'Page Up' at the top right and 'Page Down' at the bottom right. Three callout boxes with red arrows point to specific elements: one points to the 'Obtain an IP address automatically (DHCP)' checkbox, another points to the 'Obtain DNS server address automatically (DHCP)' checkbox, and a third points to the 'Cardiograph Device Name' field.

Enable DHCP by selecting this checkbox:

Note
Verify **IP Address**, **Net-Mask**, and **Gateway** fill in automatically. If not, check network cable.

Enable this checkbox to obtain a DNS Server address by DHCP.

Verify the **Cardiograph Device Name** displays the MAC 2000 serial number. Leave the default value.

Wired LAN Settings

Cardiograph Device Name GE_SDE0E210017NA

Serial/IP Redirector Listen Port 3039

Obtain an IP address automatically (DHCP)

IP Address 192.168.0.114

Netmask 255.255.255.0

Gateway 192.168.0.1

Obtain DNS server address automatically (DHCP)

Preferred DNS Server 192.168.0.1

Alternate DNS Server 0.0.0.0

Preferred WINS Server 0.0.0.0

Alternate WINS Server 0.0.0.0

Page Up

Page Down

If your network uses static IP addresses, configure the **Network LAN Settings** as follows:

The screenshot shows the 'Wired LAN Settings' window with the following fields and callouts:

- Cardiograph Device Name:** GE_SDE0E210017NA. Callout: Verify the **Cardiograph Device Name** displays the MAC 2000 serial number. Leave the default value.
- Serial/IP Redirector Listen Port:** 3039.
- Obtain an IP address automatically (DHCP):** . Callout: Disable DHCP checkbox.
- IP Address:** 192 . 168 . 1 . 39. Callout: Enter **IP Address** and **Netmask**.
- Netmask:** 255 . 255 . 255 . 0.
- Gateway:** 192 . 168 . 1 . 47. Callout: Enter **Gateway** address (if available). **IMPORTANT** If Gateway is unavailable, enter connectivity server IP address.
- Obtain DNS server address automatically (DHCP):** .
- Preferred DNS Server:** 0 . 0 . 0 . 0.
- Alternate DNS Server:** 0 . 0 . 0 . 0.
- Preferred WINS Server:** 0 . 0 . 0 . 0.
- Alternate WINS Server:** 0 . 0 . 0 . 0. Callout: If a domain was entered on the **Shared Directory Settings**, enter valid DNS and WINS addresses. Otherwise, leave the addresses set to zero (0).

After the network settings have been saved, select **Save** and proceed to "Configuring the PDF Report Name" on page 8.

Configuring a Wireless Network

On the **Communication Setup** window, page down until the **Wireless LAN Settings** page is displayed. The settings vary depending on whether your network uses DHCP or static IP addresses.

If your network uses DHCP, configure the **Wireless LAN Settings** as follows:

The screenshot shows the 'Wireless LAN Settings' window. At the top, there are fields for 'Cardiograph Device Name' (SE_S3Q13360024PA) and 'Serial/IP Redirector Listen Port' (9001). Below these are two checked checkboxes: 'Obtain an IP address automatically (DHCP)' and 'Obtain DNS server address automatically (DHCP)'. To the right of these checkboxes are the following settings: IP Address (192.168.0.114), Netmask (255.255.255.0), Gateway (192.168.0.1), Preferred DNS Server (192.168.0.1), Alternate DNS Server (0.0.0.0), Preferred WINS Server (0.0.0.0), and Alternate WINS Server (0.0.0.0). The window has 'Page Up' at the top right and 'Page Down' at the bottom right. Two callout boxes on the left provide instructions: the top one points to the first checkbox and says 'Enable DHCP by selecting this checkbox: Note Verify IP Address, Net-Mask, and Gateway fill in automatically.'; the bottom one points to the second checkbox and says 'Enable this checkbox to obtain a DNS Server address by DHCP.'

Cardiograph Device Name	SE_S3Q13360024PA
Serial/IP Redirector Listen Port	9001
<input checked="" type="checkbox"/> Obtain an IP address automatically (DHCP)	
IP Address	192.168.0.114
Netmask	255.255.255.0
Gateway	192.168.0.1
<input checked="" type="checkbox"/> Obtain DNS server address automatically (DHCP)	
Preferred DNS Server	192.168.0.1
Alternate DNS Server	0.0.0.0
Preferred WINS Server	0.0.0.0
Alternate WINS Server	0.0.0.0

If your network uses static IP addresses, configure the **Wireless LAN Settings** as follows:

Wireless LAN Settings Page Up

Cardiograph Device Name: GE_SJQ13360024PA
 Serial/IP Redirector Listen Port: 3001

Obtain an IP address automatically (DHCP)

IP Address: 192 . 168 . 1 . 39
 Netmask: 255 . 255 . 255 . 0
 Gateway: 192 . 168 . 1 . 47

Obtain DNS server address automatically (DHCP)

Preferred DNS Server: 0 . 0 . 0 . 0
 Alternate DNS Server: 0 . 0 . 0 . 0
 Preferred WINS Server: 0 . 0 . 0 . 0
 Alternate WINS Server: 0 . 0 . 0 . 0

Page Down

Callouts:

- Disable DHCP checkbox. (points to Obtain an IP address automatically (DHCP))
- Enter **IP Address** and **Netmask**. (points to IP Address and Netmask fields)
- Enter **Gateway** address (if available). (points to Gateway field)
- IMPORTANT**
If Gateway is unavailable, enter connectivity server IP address. (points to Gateway field)
- If a domain was entered on the **Shared Directory Settings**, enter valid DNS and WINS addresses. Otherwise, leave the addresses set to zero (0). (points to DNS and WINS fields)

Regardless of whether you configured the settings for DHCP or static IP addresses, page down until you reach the **Wireless Network Settings** page. Configure the wireless authentication and encryption settings as follows:

Wireless Networking Settings Page Up

Enable Wireless LAN

Network Name (SSID): DCAR-354

Authentication: WPA2-PSK

Encryption: AES

Key Index: 1

Key: systemstesting

Page Down

Callouts:

- Check this checkbox. (points to Enable Wireless LAN)
- Enter **Network Name (SSID)** (points to Network Name (SSID) field)
- Select **Authentication** type (points to Authentication dropdown)
- Select **Encryption** level (points to Encryption dropdown)
- Enter wireless password (points to Key field)

After the network settings have been saved, proceed to “[Configuring the PDF Report Name](#)” on page 8.


Configuring the PDF Report Name

The default naming convention for PDF Reports on the MAC 2000 system consists of the following components:

product_version_serial_ECGmode_cartID_creationdate.pdf

You can customize this convention using up to eight building blocks available on the **PDF Naming Settings** screen. To access this screen, select **Main Menu > System Configuration > Basic Setup**, then page down until you reach the **PDF Naming Settings** page.

Configure the PDF naming convention as follows:

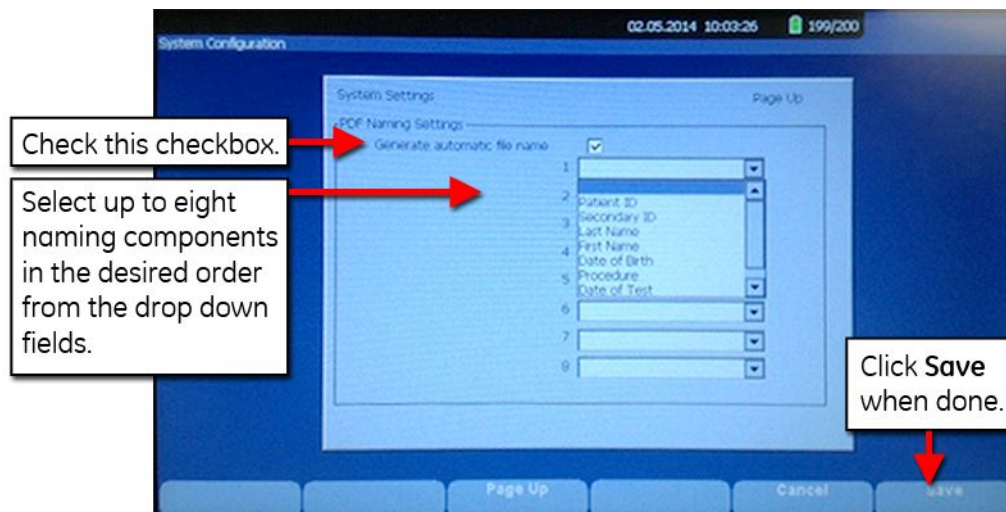After the PDF Report naming convention has been saved, proceed to “[Configuring Automatic Export to Shared Folder](#)” on page 9.

Configuring Automatic Export to Shared Folder

You can configure the MAC 2000 system to automatically export PDF Reports to the shared directory configured earlier. To so, go to **Main Menu > System Configuration > Resting ECG Setup** and page down until you reach the **Resting ECG Settings** page.

Configure the automatic export feature as follows:

Resting ECG Settings

Page Up

10s ECG Report Format: 4x2.5x3_25_R1

Detailed Results Report Format:

Report Copies: 1

Print Interpretation:

Auto Store ECG:

File Manager Sort by: Date

Auto Transmit ECG:

Delete after Transmission:

Print Transmission Log:

Auto Export ECG:

Auto Execute Single Order:

Page Down

Select **PDF** from this dropdown list.

When you are done, click **Save**. You are now ready to begin using PDF Reports.

Publication Information

The information in this manual applies only to **MAC™ 2000 ECG Analysis System** version 1.1. It does not apply to earlier product versions. Due to continuing product innovation, specifications in this manual are subject to change without notice.

MAC is a trademark owned by GE Medical Systems *Information Technologies*, Inc., a General Electric Company going to market as GE Healthcare. All other trademarks contained herein are the property of their respective owners.

Revision History

The document part number and revision are on each page of the document. The revision identifies the document's update level. The revision history of this document is summarized in the following table.

Revision	Date	Description
A	15 May 2015	Initial release.

Related Documentation

The following documents provide additional information that may be helpful in the installation, configuration, maintenance, and use of this system.

Documents Related to 2083129-009 MARS V8 to V8 SP4 Upgrade Instructions

Part Number	Document Title
2040396-104	LAN Option Installation and Troubleshooting Guide

To access this and other GE Healthcare Diagnostic Cardiology manuals, go to the Common Documentation Library (CDL), located at www.gehealthcare.com/documents, and click **Cardiology**.

Service Manual Language Information

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